


Comfort3D

powered by ORB INNOVATIONS™

Get started in 5 minutes

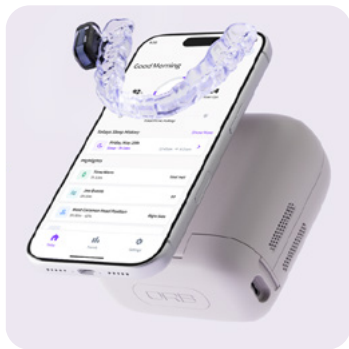

A better night's sleep starts here

- 1 Charge your device**
Charge in the case until the light is solid green
- 2 Download the app**
Tap the ORB Sleep app icon to download the app.
- 3 Pair appliance**
Open the ORB Sleep app and follow the on-screen instructions to pair your device.



**Tap to download
the ORB Sleep app**

Opens the App Store or
Google Play



What's in the box



1x Comfort3D
Powered by ORB Innovations™



1x ORB Sleep Charging Case



1x USB-C Charging Case



1x Clamshell Storage Case

Device status at a glance



Charging

Green light- slow pulse



Fully Charged

Green light- solid



Pairing Mode

Blue light- slow pulse



Connect to App

Red light- slow blink

Using your device

- ✓ Ensure the device is charged. Charge during the day, so it's always ready for your next night.
- ✓ Ensure the red light is **not** blinking. If so please connect to the ORB Sleep app **before** recording a night's sleep.
- ✓ Insert the device into your mouth and go to sleep. Recording starts automatically when you fall asleep and stops as soon as you wake up.
- ✓ Follow the guidance in the ORB Sleep app.

Sync after use

After each use, open the ORB Sleep app to sync your data. Keep your phone nearby until the sync is complete.

Need help?

Visit help.orbsleep.com
or contact support in the ORB Sleep app.

ORB
SLEEP